TRC-50DX

Retinal Camera





TOPCON'S PREMIER MYDRIATIC RETINAL CAMERA





Experience the Advantages of TRC-50DX

Rapidly developing technical advancements in digital cameras has led to retinal imaging with higher resolutions and newer capturing methods. Topcon's TRC-50DX retinal cameras incorporate state-of-the-art sensors that enhance versatility, image quality and operational ease. The TRC-50DX offers multimodal fundus imaging including color, redfree and fluorescein angiography (FA) while the TRC-50DX (Type IA) can also perform indocyanine green (ICG¹) and fundus autofluorescence (FAF¹).

Optional filters can let the operator add or exchange them to accommodate different procedures.

Features



High Quality Fundus Image



Multimodal Fundus Imaging (Color, Red-free, FA, FAF¹ and ICG¹)



Mydriatic Retinal Camera



Comprehensive Fundus Imaging Tool



High Resolution Imaging Sensors



Connect with IMAGEnet® 6

Advantages

* Image courtesy of Adachi Keiyu Eye clinic, Japa

Fundus Autofluorescence

Fundus autofluorescence (FAF¹) is a non-invasive technique to acquire images of the retina by visualizing the autofluorescence of lipofuscin within the retinal pigment epithelium (RPE). The Topcon-exclusive, Spaide FAF filters² are optimized for the observation of lipofuscin deposits in the retina as they require 40% less illumination than conventional FAF¹ filters³. As an added benefit, Spaide filters do not stimulate fluorescein or ICG¹ and images can be taken during or post-angiography.

Easy to Operate

An intuitive, touch-screen control panel simplifies its use, allowing the operator to better concentrate on the patient. Ergonomically located adjustment knobs make the capture process effortless and comfortable.



Flash Intensity Control


The Flash Intensity Control has 21 levels and can be used in full or half steps. Sensitive membrane buttons permit total control of image brightness and are illuminated for use in subdued light.


*Light Source: For photography: Max300ws Xenon



Working Distance Dots

Green alignment dots provide easy confirmation of working distance of the objective lens alignment. These dots are helpful for pre-injection positioning in fluorescein angiography (FA), red-free and color fundus photography.

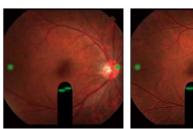

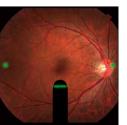

High and Wide Eyepiece Viewing

The eyepiece of the TRC-50DX is designed for high and wide eye distance viewing making alignment and focusing fast, easy and comfortable.



Software Configuration

Sensor Specifications & Product Line-up

IMAGEnet® 6

IMAGEnet 6 is a comprehensive digital imaging system capable of acquiring, processing, displaying and saving digital images obtained by a variety of Topcon devices.

IMAGEnet 6 allows the camera to detect the filter settings and CCD camera being used by the TRC-50DX and automatically sets the capture screen to the corresponding procedure.




DICOM

Full DICOM compliance allows IMAGEnet 6 to communicate with EMRS and servers, e.g. PACS, save data to remote locations, operate with modality work lists and save images in DICOM format.

Ez Capture

Ez Capture is the basic software used to capture and view images with the TRC-50DX. The live view feature within Ez Capture allows for direct observation of the retina in real time while performing ICG studies. You can easily operate ICG capture and direct view simultaneously on one monitor.

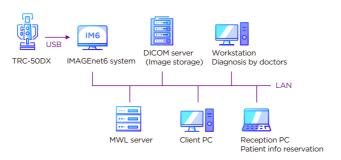



Product Line-up

	Color	Red-free	FA	FAF ¹	ICG ¹
TRC-50DX	•	•	•	-	-
TRC-50DX (Type IA)	•	•	•	•	•

1. FAF and ICG are only available on the TRC-50DX (Type IA)



	TRC-50DX	TRC-50DX (Type IA)		
Capture Mode	Color/Red-Free/FA	Color/Red-Free/FA/FAF ¹ /ICG ¹		
Auto Fluorescence	- 0			
Angle of Coverage	50°/35°/20°			
Photographic Magnifications	1.84x at 50°, 2.45x at 35°, 4.28x at 20°			
Working Distance	39.0 mm			
	0 : -10D to +6D			
Diopter Compensation Range for	- : -23D to -9D			
Patient's Eye	+ : +5D to +23D			
	A : +22D to +41D			
Diopter Compensation at Finder	-6D to +5D			
Optical Head Tilt	Upper 15° / Lower 10°			
Power Supply	100-120V/ 200-240V / 50-60 Hz			
Power Consumption	1500VA			
Dimensions / Weight	340 mm(W) x 505 mm(D)x 506-715mm(H) / 35 Kg			

Sensor Information

	Color Sensor	Monochrome Sensor
Resolution	12.3MP	5MP
Image Sensor	1.1 inch	2/3 inch

1. FAF and ICG are only available on the TRC-50DX (Type IA)

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

Not all products, services or offers are approved or offered in every market, and products vary from one country to another. Contact your local distributor for country-specific information and availability

IMPORTANT In order to obtain the best rest instructions prior to operation. In order to obtain the best results with this instrument, please be sure to review all user

TOPCON CORPORATION

75-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8580, JAPAN. Phone: +81-(0)3-3558-2522/2502 Fax: +81-(0)3-3965-6898 www.topcon.co.jp

EC REP TOPCON EUROPE MEDICAL B.V.

Esseban 11, 2908 LJ Capelle a/d IJSVE Esseban 11, 2908 LJ Capelle a/d IJSSel P.O. BOX 145, 2900 AC Capelle a/d IJSSel THE NETHERLANDS Phone: +31 -(0)10-4585045 E-mail: medical@topcon.com www.topconhealthcare.eu

TOPCON HEALTHCARE SOLUTIONS EMEA OY

HQ & PRODUCT DEVELOPMENT Saaristonkatu 23, 90100 Oulu, FINLAND Phone: +358-20-734-8190 E-mail: thsemea.sales@topcon.com www.topconhealth.eu

TOPCON DANMARK
Praestemarksvej 25, 4000 Roskilde
DANMARK
Phone: +45-46-327500 Fax: +45-46-327555 E-mail: info@topcon.dk www.topconhealthcare.eu

TOPCON SCANDINAVIA

Neongatan 2, P.O.Box 25, 43151 Mölndal SWEDEN Phone: +46-(0)31-7109200 Fax: +46-(0)31-7109249 E-mail: medical@topcon.se www.topconhealthcare.eu

TOPCON ESPAÑA S.A.

HEAD OFFICE Frederic Mompou, 4, 08960 Sant Just Desvern Barcelona, SPAIN Phone: +34-93-4734057 Fax: +34-93-4733932 E-mail: medica@topcon.es www.topconhealthcare.eu

TOPCON ITALY

Viale dell' Industria 60, 20037 Paderno Dugnano, (MI) ITALY Phone: +39-02-9186671 Fax: +39-02-91081091 E-mail: info@topcon.it www.topconhealthcare.eu

TOPCON FRANCE MEDICAL

1 rue des Vergers, Parc Swen, Bâtiment 2, 69760 Limonest, FRANCE Phone: +33-(0)4-37 58 19 40 Fax: +33-(0)4-72 23 86 60 E-mail:topconfrance@topcon.com www.topconhealthcare.eu

TOPCON DEUTSCHLAND MEDICAL

Hanns-Martin-Schleyer Strasse 41, D-47877 Willich, GERMANY Phone: (+49)2154-885-0 Fax: (+49)2154-885-177 E-mail: info@topcon-medical.de www.topconhealthcare.eu

TOPCON POLSKA SP. Z. O. O.

UNUM POLISKA SP. Z. O. O.
UN WARSZAWSKA 23, 42-470 Siewierz POLAND
Phone: +48-(0)32-671-50-45
Fax: +48-(0)32-671-34-05
E-mail: info@topcon-polska.pl
www.topconhealthcare.eu







TOPCON (GREAT BRITAIN) MEDICAL LIMITED Topcon House, Kennet Side, Bone Lane,

Newbury, Berkshire RG14 5PX UNITED KINGDOM Phone: +44-(0)1635-551120 Fax: +44-(0)1635-551170 E-mail: medical@topcon.co.uk www.topconhealthcare.eu

TOPCON IRELAND MEDICAL Unit 292, Block G, Blanchardstown, Corporate Park 2 Ballycoolin Dublin 15, D15 DX58, IRELAND Phone: +353-12233280 E-mail: medical.ie@topcon.com www.topconhealthcare.eu

